

Topi-CLICK 35 DoseCheck White

The Topi-CLICK® 35 DOSECHECK™ topical dosing dispenser provides the same advanced technology as the Topi-CLICK® 35 and offers EXTRA assurance with graduated marks so the dose starting and stopping points can be easily verified and recorded. Topi-CLICK® 35 DoseCheck™ initial start point should begin when the bottom of the plunger is at the 30 mL graduated mark.

Consistent & Accurate Dose™

- Topi-CLICK® dispensers proven most accurate among others tested by ARL
- Eliminates user-variability
- Consistent automated dosing throughout treatment
- Allows for easy recording of numerical volumes dispensed

Built-in Applicator Pad

- Reduces the risk of cross-contamination to others

CLICK-Assist™ Technology

- Clear Audible CLICKs
- Built-in Visual CLICK points
- Strong Tactile Sensations at each CLICK point

Viscosity Assurance™

- Accuracy NOT affected by varying formulations
- Accuracy NOT affected by varying bases

Made of FDA grade materials | BPA and Phthalate free | UV block and is Recyclable

Designed with graduated marks so the dose starting and stopping points can be easily verified and recorded.

Dispenser Volume: Up to 34mL

Delivery Volume: 30mL (based on beginning initial start point at the 30mL graduated mark)

Dispense per CLICK: 0.25mL

Approximate Residual: 3.50mL (Residual remaining after plunger reaches the “0” graduated mark)

Approximate Priming: 0.50mL or two CLICKs

Doses Provided: The Topi-CLICK® 35 DoseCheck™ will provide approximately 30 mL or 60 doses (at 2 CLICKs per dose or 120 CLICKs total) between the “30” and “0” graduated marks. 1 mL of cream can still be dispensed from the DoseCheck after reaching “0”. The 1 mL buffer space is there to ensure accurate visual dose tracking and dispensation is maintained for the patient.

