

**NEW!**

Topi-CLICK

**MICRO<sup>®</sup>**

Accurate & easy topical  
micro-dosing applicators  
for humans and pets

**0.05 mL  
Per  
CLICK**



*Choose Topi-CLICK Micro<sup>®</sup> for  
The Best User Experience<sup>™</sup>*



**Topi-CLICK  
Micro<sup>®</sup>**  
(0.05 mL/CLICK)

- ✓ Automated, metered topical dosing provides an accurate and consistent dose every time with built-in applicators
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Built-in REFILL Badge<sup>™</sup> helps the pet owner reorder medication on-time
- ✓ Innovative TRIAD-shape allows for precision handling



Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️.



**DOSELOGIX**

Topi-CLICK Micro<sup>®</sup> Patent Pending    Topi-CLICKVetUser\_071519  
**DoseLogix.com | 877.870.8448 | info@DoseLogix.com**

Copyright © 2019 DoseLogix, LLC All Rights Reserved.

# Topi-CLICK MICRO®

## Veterinary Patient Instructions




### Topical Micro-dosing Application

- ✓ Automated, metered topical dosing provides an accurate and consistent dose every time
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative TRIAD shape allows for precision handling
- ✓ Built-in REFILL Badge™ helps pet owners reorder medication on-time



**STEP 1:**  
Unscrew the cap of the Topi-CLICK Micro® and set aside for recapping after use.

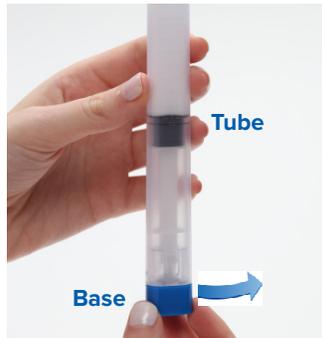

**STEP 2:**  
Holding the Micro® in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile, and Visual CLICK points of the triad shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure dose is fully dispensed.



**STEP 3:**  
Apply topical medication with built-in applicator as directed by veterinarian or pharmacist. **Continue rubbing until preparation has completely saturated the prescribed areas through your pet's fur.** **IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.



**STEP 4:**  
Screw on cap after each dose to prevent evaporation. Be sure cap is secured completely.

Innovative  
TRIAD Shape  
Allows Precision  
Application  
Control

Each turn/CLICK of the  
base dispenses 0.05 mL of  
medication accurately.



### The Refill Badge™ reminder system helps pet owners reorder in a timely manner.

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

*Note: Follow instructions on prescription label before use. Topi-CLICK Micro® is not re-fillable.*

Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️